



# Medline Industries, LP

## **IMMEDIATE ACTION REQUIRED**

### Medline Hudson RCI Medium Concentration Oxygen Masks

11/21/2025

Dear Valued Medline Customer:

Medline Industries, LP is issuing a recall for specific item(s) and lot(s) of Medline Hudson RCI Medium Concentration Oxygen Masks (item number(s) HUDRHO41U, HUD1930, HUD1035, and HUD1042). Medline has received multiple complaints regarding the tubing disconnecting from the oxygen mask. If the tubing were to disconnect from the oxygen mask during use, there may be a delay in patient care and the patient may experience shortness of breath leading to hypoxia, which may require medical intervention to prevent further respiratory deterioration. Affected product is to be destroyed. Please be advised that there may be item number variations based on the unit of measure ordered by your facility, indicated by an H appended to the item number(s) included with this communication. Note, the product labeling will reflect the base item number (e.g., HUD1930). The product label will not include an indication of any individual unit of measure (e.g., HUD1930H) which may have been purchased by your facility. There is no product difference between variations in units of measure. Please refer to the enclosed list for all affected item number(s) and affected lot number(s).

Adverse reactions or quality problems experienced with the use of this product may be reported to the FDA's MedWatch Adverse Event Reporting program either online, by regular mail or by fax and to Medline Industries, LP.

#### **REQUIRED ACTION:**

1. Immediately check your stock for the affected item number(s) and the affected lot number(s) listed on the enclosed response form. Quarantine all affected product. Upon completion of the below actions, please destroy affected product.
2. Please complete and return the enclosed response form listing the quantity of affected product you have in inventory. Even if you do not have any affected product in inventory, please complete and return the response form.
3. Your account will receive credit once the response form is submitted, if applicable.
4. If you are a distributor, or have resold or transferred this product to another company or individual, you are required by FDA regulations to notify them of this recall communication. Please request that customers destroy any affected product. You should include your customers' quantities on your response form.

If you have any questions, contact the Recall Department at 866-359-1704 or [recalls@medline.com](mailto:recalls@medline.com).

Please accept our sincere apologies for any inconvenience this may have caused. We, like you, place the health and safety of your patients first and foremost.

Sincerely,

*Karin Johnson*

Karin Johnson  
Product Recall Specialist Sr.

**Recall Ref#: R-25-234**

Confidential – For intended recipients only. Do not Distribute.

**Medline Industries, LP.** Three Lakes Drive      Toll Free: 866.359.1704      Email:  
Northfield, IL 60093-2753      Website: medline.com      [Recalls@Medline.com](mailto:Recalls@Medline.com)  
Fax: 866.767.1290